

Band Tourniquet

An easy to use, band type tourniquet designed with smooth texture for patient comfort.



User Instructions



Step by Step Guide

<i>Model</i>	<i>Size</i>	<i>Length</i>	<i>Colour</i>
AMTL-2803	Adult	440mm	

1. Holding each end, place the tourniquet under the patients arm, stretch the tourniquet and bring each end around to the front
2. Cross over each end at the front whilst maintaining the tension and use thumb and index finger to hold the cross over together
3. Using the other hand, create a loop at one end of the tourniquet and pass this end underneath the tourniquet at the front of the patients arm and release
4. To remove, pull the loop end of the tourniquet