


BV

Safety Lancet

An easy to use, ergonomic lancet designed with safety to protect healthcare professionals and carers from needle stick injuries.



User Instructions

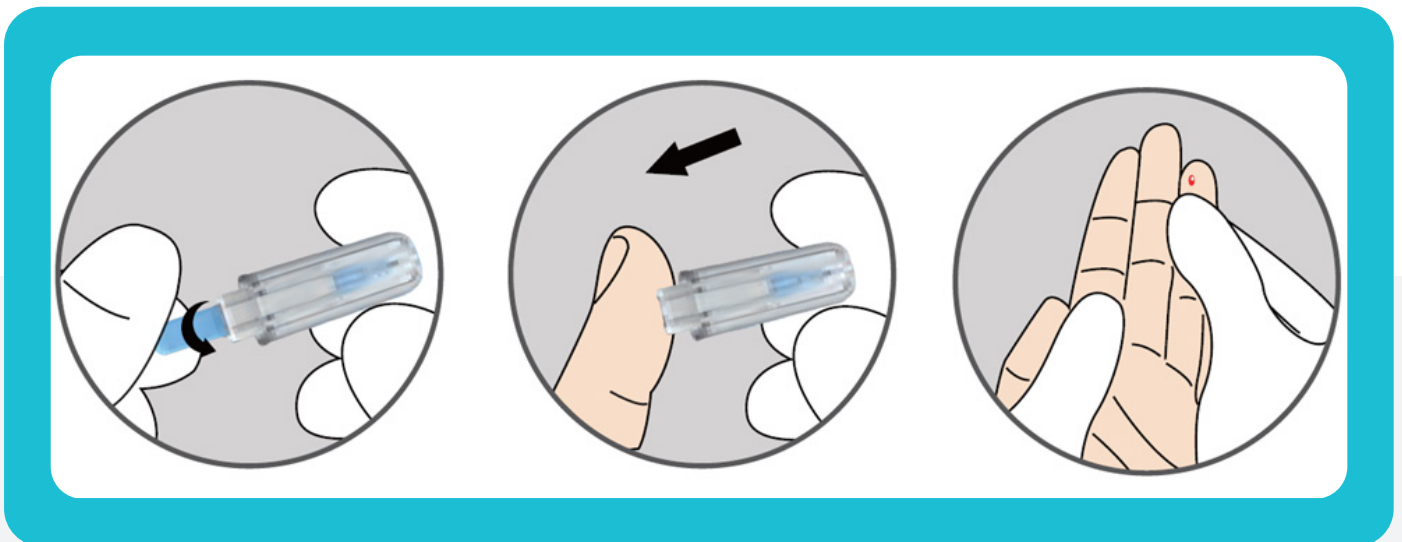

Step by Step Guide

Model	Gauge	Depth	Colour
AMTL-BV1826	26G	1.8mm	Yellow
AMTL-BV1828	28G	1.8mm	Blue
AMTL-BV1830	30G	1.8mm	Orange

Please clean and dry a test site before commencing

1. Twist and remove the protective cap from the lancet
2. Place the small grey end of the lancet on the test site
3. Gently push the lancet down against the test site to activate the lancet
4. Once used, discard the lancet into a sharp container