


# Twist Cap Lancet

An easy to use, universal lancet designed to fit most lancing devices.



## User Instructions

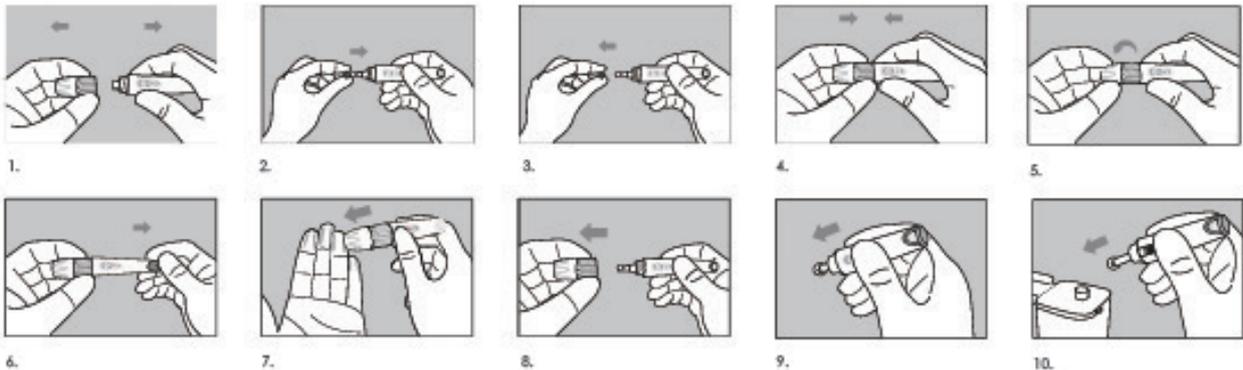

## Step by Step Guide

Model	Gauge	Depth	Colour
AMTL-2800	28G	Set by lancing device	
AMTL-3000	30G	Set by lancing device	

**\*Please clean and dry a test site before commencing\***

1. Open by rotating/unplugging the pen cap
2. Insert a new, sterile lancet into the needle holder until it stops
3. Twist off the protective cap of the lancet
4. Tighten the pen cap back onto the pen body
5. Twist the cap to select puncture depth from deep to shallow, counterclockwise 6→5.5→5→4.5→4→3.5→3→2.5→2→1.5→1  
- due to the different thickness of a humans skin, choose the depth according to personal needs to reduce the discomfort of blood sampling
6. Pull the rear sliding barrel back until it clicks and you will see the color change inside the release button
7. Press the cap to the side of the fingertip and press the release button to puncture the skin
8. After the blood collection is complete, rotate/unplug the pen cap
9. Push the used lancet needle into the middle of the exposed side of the plastic cap
10. Push the ejector forward to eject the used lancet into an appropriate container