

# Button Tourniquet

A easy to use, button type tourniquet designed with easy single hand removal for patient safety.



## User Instructions



## Step by Step Guide

<i>Model</i>	<i>Size</i>	<i>Length</i>	<i>Colour</i>
AMTL-2801	Adult	366mm	
AMTL-2802	Paediatric	250mm	

1. Holding each end, place the tourniquet under the patients arm and bring each end around to the front
2. To avoid over tightening and skin damage when applying, support the button with two fingers
3. To create tension keep the button end of the tourniquet close to the arm and pull the other end upwards then over the button
4. Gently press the button end of the tourniquet through the hole to secure tension
5. To remove, place index and middle fingers underneath either side of the excess tourniquet and slide fingers down either side of the button
6. Use thumb to hold down on button and rotate hand 90 degrees upwards and gently push button through hole to release.